

MELCO Catalog No. TS-2

DONOR TUBE STRIPPER

DIRECTIONS FOR USE:

- 1.** Hold needle hub in one hand and donor tube stripper in the other hand.
- 2.** Insert needle end of donor tube between the two rollers of the stripper.
- 3.** Squeeze stripper and roller from needle hub toward the tubing until rollers touch top of bag.
- 4.** Release stripper and remove from tubing.

www.melcowire.com

**Melco
Engineering**



(888) MELCO-88

P.O. Box 8907 • Calabasas, CA 91372-8907

Email: medical@melcowire.com