

MELCO Catalog No. TS-2

DONOR TUBE STRIPPER

DIRECTIONS FOR USE:

- 1.** Hold needle hub in one hand and Donor Tube Stripper in the other hand.
- 2.** Insert needle end of the Donor Tube between the two rollers of the stripper.
- 3.** Close stripper and **PULL THE TUBING** until the rollers touch the top of the bag.
- 4.** Release Donor Tube Stripper and remove from the tubing.

www.melcowire.com

**Melco
Engineering**



(888) MELCO-88 • Fax: (818) 591-1073

P.O. Box 8907 • Calabasas, CA 91372-8907

Email: support@melcowire.com